Ezi-Crown®

INSTRUCTION FOR USE

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Ezi-Crown is light cured, temporary crown and bridge resin. It offers a simple way of making temporary inlay, onlay, crown and bridge.

■ Indications

1. Single crown
2. Bridge
3. Immediate implant restoration
4. Relining case

■ Composition

1. The monomer matrix of UDMA
2. The contents of Inorganic filler (Particle size 0.01~2.5 μ m) 70~75%

■ Application

1. Before the application
 - ① Check the expiration date and be cautious of the package contaminated.
 - ② Follow the instructions before using it.
 - ③ When ready to administer the procedure to the patient, remove the cap of the product.
 - ④ Please protect the oral cavity using a rubber dam.

2. Application

- ① Apply vaseline on the teeth.
- ② Cut off approximate amount of Ezi-Crown.
- ③ Apply Ezi-Crown. Press into place, shape and sculpt.

- ④ Make the occlusion. Have a patient bite softly to register occlusal surface and adjust.

- ⑤ Apply the curing light. (Curing time : 20 sec.)

- ⑥ Remove the cured Ezi-Crown from the teeth, and apply the curing light again on the inside. (20 sec.)

- ⑦ If you need polishing and trimming, do that with lab, carbide bur or silicone point.

■ Contraindications


Do not use on patients with hypersensitivity history or allergies to Methacrylate-based resins.

■ Cautions

1. For dental use only.
2. It is not used in patients with a history of rash and hypersensitivity to dermatitis.
3. This product is used only for oral tissue. Be careful not to get into your eyes.
4. If you contact your skin or come into eyes, wash them several times with clean water and consult a specialist if necessary.
5. This product is prohibited for use other than its intended use. And the use of non-professionals is prohibited.
6. Observe the procedure and the precautions.
7. Keep it away from a reach of non-dental users including children, senior citizen and other.
8. Do not use after expiration date.
9. There is a risk of product damage, so don't give excessive impact.
10. Since it hardens by light, immediately place the container and the light-blocking aluminum foil pouch after use.

■ Storage

1. Store with the cap closed in the provided polypropylene black case.
2. Store at a temperature between 1~30°C.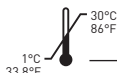

■ Contents

- # 701-1 Ezi-Crown Refill A1 15g x 3ea
- # 701-2 Ezi-Crown Refill A2 15g x 3ea
- # 701-3 Ezi-Crown Refill A3 15g x 3ea
- # 702-1 Ezi-Crown Single A1 15g x 1ea
- # 702-2 Ezi-Crown Single A2 15g x 1ea
- # 702-3 Ezi-Crown Single A3 15g x 1ea
- # 703-1 Ezi-Crown A1 15g x 2ea
- # 703-2 Ezi-Crown A2 15g x 2ea
- # 703-3 Ezi-Crown A3 15g x 2ea

■ Shelf-life

3 years from the date of manufacture

EC REP

KTR EUROPE GMBH

Mergenthalerallee 77, 65760 Eschborn, Germany
Phone : +49-6196-887170

SYMBOLS

: Use-by date	: Batch code
: Caution	: Keep Dry
: Manufacturer	: Date of Manufacture
: Authorized Representative in the European Community	
: Temperature limit	: Consult instruction for use
: Keep away from sunlight	: Catalogue number
: Model number	: Medical device
: Unique device identifier	: Non-sterile device

MEDICLUS CO., LTD.

No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Republic of Korea
Tel. +82-43-211-2877 / Fax. +82-43-211-2866
E-mail : sales@mdclus.com