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EN

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: sodium hypochlorite (5.25% of active chlorine), water Ph. Eur., surfactants.

INTENDED USE

Liquid for root canal rinsing. Removal of dead pulp residues during mechanical disinfection of the root canal. Cleans the root canal, including the syrnaxial layer to expose the distal tubule orifices prior to filling the canal.

Prevents discoloration of the teeth that might occur after filling the non-rinsed root canal.

Adapter with a plug for taking the liquid preparations into the syringe

- UnscREW the bottle cap.
- Slide the adapter on the bottle.
- Remove the blue plug, screw the end of the syringe into the adapter's opening. Turn the bottle upside down and pull the liquid into the syringe, while simultaneously unscrewing the adapter. Next, put the bottle upright on the table top and gently unscrew the syringe from the adapter. Use luer-lock syringes with a rubber plunger.
- After each use, close the adapter's opening with a blue plug.
- For transport, remove the adapter and close the bottle with a bottle cap.

INSTRUCTIONS FOR USE

Insert CHLORAXID 5.25% EXTRA into the root canal using the syringe with a needle with a side channel and start rinsing.
Sodium hypochlorite in the product inactivates immediately in contact with organic substances. Therefore, to compensate, the amount of the preparation should be increased to about 10 ml per canal.

CONTRAINDICATIONS
Do not use on patients who are hypersensitive to the product components.
CAUTION! Flushing root canals: separate the sodium hypochlorite flush and the chlorhexidine digluconate flush with a normal saline flush to prevent a brownish sediment from forming. Thoroughly flush and rinse.

PRECAUTIONS

During root canal preparation do not use a too heavy stream of the liquid because it can cause the liquid to penetrate the periapical tissues and damage them, resulting in pain and swelling of soft tissues.
Special care should be taken when the product is applied into perforations. May cause severe skin burns and eye damage.
In case of contact rinse immediately with plenty of water and seek medical advice. In case of consumption do not induce vomiting. Drink plenty of water. Seek medical advice.
Use rubber-dam while working with the product. The use of personal protective equipment (glasses, protective visor, gloves) is recommended.

DISPENSER IS NOT INTENDED FOR LIQUID STORAGE!

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in original container at 2-8°C (refrigerate). Protect from sunlight. Keep out of reach of children.
Once container is open, the expiry date does not change if the container is securely closed after each use.

TRANSPORT

The product can be shipped for a maximum of 10 days under 40°C. Short distance shipping and handling within the recommended temperature range does not affect the safety of use and efficacy of the product before its expiry date.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to manufacturer.

PACKAGING

Bottle containing 200 g or 400 g of the preparation, adapter with a screw cap, dispenser.

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