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EN
PLEASE READ THE FOLLOWING INSTRUCTIONS BEFORE USE
PRODUCT FOR DENTAL USE ONLY

EN
COMPOSITION:
<i>MTA+ powder:</i> calcium oxide, oxides of silicon, iron, aluminium, sodium, potassium, bismuth, magnesium, zirconium; calcium phosphate.

INTENDED USE

The product is intended for dental treatment as a material for filling and remineralisation of root canals. The compound cures in 6 to 10 minutes after mixing with water. The high content of calcium ions remineralises the tooth (restores calcium and calcium compounds support tissue) and helps with pulp perforation and intracanal resorption. MTA+ is especially recommended in:

- root canal wall perforation
- resorption of the root canal
- filling of the posterior root apex
- direct coating of the pulp
- pulp amputation
- treatment of teeth with interrupted root development.

MTA+ is not suitable as a standalone permanent root canal filling.
INSTRUCTIONS FOR USE

Cut off the *MTA+ liquid* pipette tip. Move the entire contents of the *MTA+ powder* glass vial with 1 to 2 drops of the *MTA+ liquid* on a mixing plate. Mix for ca. 30 s until the compound reaches a consistency of moist sand. If the compound mix is too thick or too brittle, add one more drop of the liquid (or distilled water). Place the compound at the target site with an applicator. Exposure to moisture, including blood, will reduce the curing time. One container of the *MTA+ powder* contains a single dose. Once mixed with the *MTA+ liquid* (or deionised water) the mixed dose must be used within 10 minutes. The compound cures and hardens afterwards. Use the contents of the *MTA+ liquid* container within 12 months after opening.

CONTRAINDICATIONS

Do not use in patients who are hypersensitive to any of the ingredients. Do not use in inflammatory conditions. Acidic pH values disrupt setting of MTA+.

PRECAUTIONS

The product can cause eye damage. Irritating to the mucous membranes of the oral cavity and skin. In case of contact, rinse with plenty of water and contact a doctor immediately. If swallowed, do not induce vomiting Drink plenty of water. Contact a doctor immediately. Use rubber dams, disposable gloves and safety goggles when working with the product.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

STORAGE

Store in a tightly closed original container below 25°C.

Protect from moisture content. Store below 6-10 Minutes. Do not use after expiry date of each of children. The expiry date is on the direct packaging.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly or returned to the manufacturer.

PACKAGING
Glass vial containing 0.14 g of the <i>MTA+ powder</i> ; pipette containing 1 ml of the <i>MTA+ liquid</i> .

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