CE

LASCOD

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APPLICATIONS

Preliminary impressions for the two-step putty/wash and base for the sandwich technique, reproduction of peripheries for full dentures relevant for bite-taking. The impression can be electroplated.

PREPARATION

Dry the areas to be reproduced and prevent further inflow of liquid into them. Use Enersyl catalyst exclusively in the following amounts: for 12 g of silicone, corresponding to about a level scoopful, 0.3 g of catalyst corresponding to a thread approximately 3.5 cm long (the diameter of the scoop) extended without stretching (1). Mix the catalyst and silicone vigorously until a uniformly colored mass is obtained (2). Place the mixed material on the impression tray (3) and insert it into the mouth (4).

Preliminary impression

Wash impression

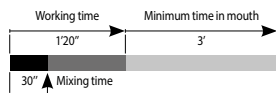
APPLICATIONS

Silicone indicated for the two-step putty/wash and for the sandwich technique, syringeable. The impression can be electroplated.


PREPARATION

Dry the areas to be reproduced and prevent further inflow of liquid into them. Use Enersyl catalyst exclusively in the following amounts: for 10 g of silicone, 0.8 g of catalyst paste. The mixing guide can be used to facilitate dispensing the catalyst. Extrude the Silaxil into the area indicated between the two lines from the extreme left to the right. Do so in such a manner that the paste is extruded widely enough to meet the two parallel lines without exceeding them. Dispense the catalyst using the same procedure (5). The oblique lines correlate the amount of silicone with the required amount of catalyst. With a steel spatula, move the Silaxil and catalyst towards the centre of the guide and mix vigorously to eliminate any air bubbles in order to obtain a uniform color (6). Place the mixed material in the impression tray with a spatula or with a syringe in the case of elastomers (7) and insert it into the mouth (8).

PRELIMINARY IMPRESSION

WASH IMPRESSION

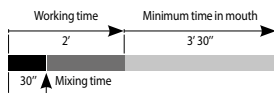TECHNICAL DATA	Preliminary impression	Wash impression	
ISO 4823	Type 0 - Putty	Type 2 - Regular Body	Type 3 - Light Body
Mixing time	30"	30"	30"
Working time *	1' 20"	2'	2'
Minimum time in mouth	3'	3' 30"	3' 30"
Recovery from deformation	99,75%	99,9%	99,9%
Strain in compression	3,5%	4,5%	4,5%
Linear dimensional change at 24 hours	-0,15%	-0,2%	-0,2%
Detail reproduction	40µm	5µm	5µm
Hardness Shore A	53	40	40

* Recorded at 23 +/- 2°C and 50 +/- 10% relative humidity.

INDICATIONS: It is recommended not to exceed the indicated dose of catalyst. Higher temperatures or a greater amount of catalyst reduce the times indicated in the table, lower temperatures or a lower amount of catalyst increase them. After taking the preliminary impression, rinse it under running water to remove traces of saliva or blood and dry well to ensure optimal adhesion of the wash impression if this has still to be taken. The impression can be kept without particular devices. Cast the plaster model at least 15 minutes after removal from the mouth. Optimal results can be obtained with Ortoyplo, Kromotypo or Singletypo type II or IV plaster from the Lascod range. Close the packs well after use. Store in a cool and dry place. For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

WARNINGS: Use within the expiry date and for the applications described. For use, wear protective gloves, eye protection and protective clothing. Avoid contact during pregnancy and while nursing. Do not use on children. Avoid that the catalyst comes in contact with eyes and/or skin, that is inhaled or swallowed. If in eyes or on skin: rinse cautiously with water for several minutes. If swallowed, or if you feel unwell, get medical advice.

IMPRESSION TRAY: All types of rigid impression trays can be used. Use of the Lascod Exploit impression tray is recommended.

OTHER AVAILABLE SILICONES

For dental clinics: OKLUREST and GHENESYL addition silicones.

For laboratories: ERGASIL condensation silicone; ERGAMIX and XILGUM addition silicones.

GUARANTEE

LASCOD SpA guarantees the quality of its products and undertakes to replace them whenever defects in production are found within the expiry dates. LASCOD SpA will not be held responsible and will not recognize any damage that is direct, indirect, accidental or caused by incorrect use of the material. Check the suitability of the product prior to use.

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