VOCO Amaris®

CE 0482

GB

Highly aesthetic light-curing restorative in accordance with EN 24049/ISO 4049

Instructions for use

Amaris® is a light-curing, radiopaque restorative which contains 80 w/w % filler (in-dividual mass shades HT and HO 64 w/w %) in a methacrylate matrix (Bis-GMA, UDMA, TEGDMA). **Amaris®** cures under blue light (halogen and LED) and is available in syringes and Caps for direct applications. **Amaris®** is used in combination with a dentine-enamel bond.

Shades:

Base shades (opaque)

O1, O2, O3, O4, O5, O Bleach provide the translucency and chroma of natural dentine.

Enamel shades (translucent)

TL (Translucent Light), **TN** (Translucent Neutral) and **TD** (Translucent Dark) provide the translucency of natural enamel.

Special shades

HT (High Translucent) and **HO** (High Opaque) are flowables.

Indications:

- Aesthetic restoration of class III, IV and V cavities in anterior teeth
- Aesthetic restoration of class I, II and V cavities in posterior teeth
- Reconstruction of traumatically damaged anterior teeth
- Direct and indirect composite veneers and facing of discoloured anterior teeth
- Shape and shade correction for improved aesthetics
- Repairs of highly aesthetic porcelain, e.g. veneers
- Splinting of anterior teeth

Directions:

Preparation:

Clean the teeth to be treated with a fluoride-free cleaning paste. Mark occlusal contact points (posterior area); minimal separation facilitates the shaping of the approximal contacts and placement of the matrix.

Shade selection:

The components of **Amaris®** are designed to provide first-class aesthetic restorations of large anterior defects using the layer technique. The palatal-vestibular layering technique is recommended in particular.

To select a shade moisten the tooth and the shade samples and then compare the enclosed shade guide to the cleaned tooth in natural light before administering anaesthesia.

Use the cervical area of the natural tooth or the shade of the moistened dentine, as visible in the cavity, as a guide for selecting the most suitable base shade (**O1-O5**). Refer to the incisal, occlusal or approximal area of the natural tooth to select the enamel shade (**TL/TN/TD**).

Note: Begin by applying a layer of the selected opaque shade to the defect (direction oral to vestibular) in cavity classes III, IV and V. The overall impression of the restoration in terms of shade can be influenced and adjusted through the selection of translucent shades for the finishing vestibular layer. The opaque base shade can be **darkened with TD, confirmed with TN and lightened with TL**. In cavity classes I and II apply a layer of the corresponding base shade to dentine following the customary layer technique. Afterwards, apply a translucent shade as the enamel layer to finish the restoration.

With the assistance of the shade samples for the shades **HT** and **HO**, an impression of high transparency and high opacity for the individual shades can be achieved.

Using the shade **HT** (High Translucent), especially in the incisal area, allows for creating a perfectly individualised restoration.

The shade **HO** (High Opaque) can be used for masking surfaces that are discoloured, e.g. iatrogenic tooth discolouration, which is important for extensive defects.

Drying:

Securing a dry working area is essential. Use of a rubber dam is recommended.

Cavity preparation:

In order to conserve healthy tooth substance, minimally invasive cavity preparation should be carried out strictly according to the rules of adhesive filling technique. Non-carious lesions in the cervical area do not necessarily need to be prepared, through cleaning is sufficient. Clean and dry the cavity after preparation.

Lining:

Apply a calcium-hydroxide liner in proximity of the pulp and cover it with layer of a stable cement.

Bond material:

Amaris® is used according to the adhesive technique, with a dentine-enamel bond. All light-curing bonds can be used with **Amaris®**. Follow the instructions for use of the bonding material regarding preparation (etching technique) and processing.

Applying Amaris®:

Allow **Amaris®** to reach room temperature before application. For cavities in approximal areas apply a matrix. Using a translucent matrix is beneficial (smooth surfaces).

Apply the desired shade of **Amaris®** (see shade selection) and adapt it to the cavity with a suitable instrument. For fillings of more than 2 mm (shade HO: more than 1 mm), apply and polymerise in layers. Insert Caps into the opening of the dispenser following the respective instructions for use. Turn Caps in the desired direction and remove the protective cover. Apply **Amaris®** directly into the cavity by slowly and evenly pressing the levers of the handle together.

Light-curing:

Customary polymerisation devices are suitable for light-curing. The light output should be a minimum of 500 mW/cm².

The minimum curing times per layer are as follows:

Amaris® Base shades O1-O5, O Bleach	40 s
Amaris® Enamel shades TL/TN/TD	10 s
Amaris® Flow High translucent HT	10 s
Amaris® Flow High opaque HO	40 s

Hold the light emission tip of the device as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth may be reduced. Insufficient curing can lead to discolouration and complaints in the area of the pulp .

Finishing:

The filling can be finished and polished immediately after removing the shaping acids (e.g. fine diamond polish, polishing disc) with a curing product. As a final step, the margin of the filling or the entire tooth should be fluorideated.

Advice and precautions:

- No known side effects. However, sensitisation in hypersensitive patients cannot be ruled out.
- Amaris®** should not be used if the prescribed application technique cannot be used or if there are known allergies to its ingredients (Bis-GMA, UDMA, BHT, TEGDMA).
- Phenolic substances, especially eugenol- and thymol-containing preparations, interfere with the curing of filling composites. Do not use zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- Fillings exposed to occlusal forces should be checked at least once a year to detect any changes early.
- The enclosed application cannulae for use with the individual shades **HO** and **HT** should only be used for one application. Discard cannula after use and close the syringe with the protective cap.
- To prevent unwanted flow of material from individual shade syringes, slightly pull back on the piston.

Storage:

Store at temperatures between 4°C - 23°C. If refrigerated, allow **Amaris®** to reach room temperature before use. Immediately close syringes after dispensing to avoid exposure to light which can lead to polymerisation. Do not use **Amaris®** after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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Schmelzfarben (Translucent)

TL (Translucent Light), **TN** (Translucent Neutral),

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