



# MANIshine

## WHITENING OFFICE SET

MANIshine – the whitening expert  
for premium results and smiling patients!

**MADE IN GERMANY**



# MANI

**MANIShine** – the whitening expert for first-class results and smiling patients! Two-component bleaching gel in an automix cartridge with 35% hydrogen peroxide for effective whitening of individual or multiple teeth, including endodontically treated teeth.


Also included are the MANIShine Gingiva Guard and Remin Care remineralizing paste.

## INDICATIONS

- ▼ Whitening of discolored, devitalized teeth
- ▼ Whitening of teeth that are discolored due to disease, trauma, or iatrogenic causes, as an alternative to crowns or veneers
- ▼ Treatment of medication-induced tooth discoloration

---

## PRODUCT FEATURES

- ▼ Neutral pH value
  - ▼ **Fast and reliable brightening**
  - ▼ Long-lasting results
  - ▼ Gentle on teeth
  - ▼ Easy to use without running
  - ▼ No mixing required, ensuring consistent quality
  - ▼ **Safe transport and guaranteed material stability thanks to the safety clip**
- 

# STEP BY STEP

## 1 PREPARATION

Determine tooth color (comparison with Vita® color scale). Clean teeth thoroughly (Air-Flow or cleaning paste). Insert cheek/lip retractor and apply gingival protection.



Application of MANiShine Gingiva Guard.  
Polymerise for 20 seconds to cure.  
Protection for the gums.

## 2 APPLY BLEACHING GEL

Prepare the syringe (remove the cap, attach the mixing tip). Dispense the gel until it is homogeneous. Apply evenly to the areas to be bleached in a layer 1–2 mm thick (Close after usage).



## 3 EXPOSURE TIME & AMPLIFICATION

Leave Active White 35% Gel on for 10 minutes. Optional: Light amplification with plasma light or laser (note eye protection). Up to three bleaching procedures are possible during one session.



## 4 AFTER TREATMENT

Remove the gel and rinse thoroughly. Dry the surface and polish with polishing paste. Follow up with ManiShine Remin Care. Do not eat, drink, or smoke for at least 1 hour after treatment. Avoid staining foods for a few days.




After treatment, remove the guard and use MANiShine Remin Care.

# MANIShine Active White 35% IN COMPARISON

## BLEACHING

	MANIShine	Competitor 1	Competitor 2
Hydrogen peroxide concentration (mixed)	35%	38%	29%
pH value (mixed)	7,3	8,0	7,4
Color	Red	Red	Blue
With remineralization supplement	Yes	Yes	No
With additives for pain reduction	Yes	Yes	Yes
Administration	Double cartridge (automix)	Two syringes, manual mixing required	Double cartridge (automix)
Special features	Special clip (better stability, no bubbles)	No	No
Consistency: Gel (easy to use, high level of safety)	Yes	Yes	No

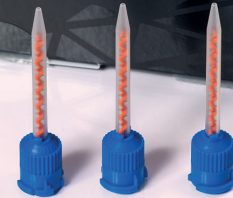

1

Protecting the gums with

**MANISHine**  
Gingiva Guard




 eIFU ↑

2

Application of

**MANISHine**  
Active White 35%




 eIFU ↑

3

Complete treatment with

**MANISHine**  
Remin Care



 eIFU ↑

# PHYSICAL DATA\*

## MANIShine Gingiva Guard

Hardness	Density	Color
4,5 HV	1,28 g/cm <sup>3</sup>	blue

\* All technical data are average values and may vary slightly in each delivery batch.



### Content

**MANIShine Gingiva Guard, 1,5 g**

**MANIShine Active White, contains hydrogen peroxide 35%, 5 g**

**MANIShine Remin Care, 2 ml**

**3 Mixing Tips**

**3 Application Tips**

**MANI MEDICAL GERMANY GmbH**

Hertha-Sponer-Straße 2  
61191 Rosbach v.d. Höhe

info@mani-germany.com  
[www.mani-germany.com](http://www.mani-germany.com)



# MANI