Any-Flex™

INSTRUCTION FOR USE

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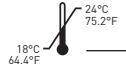
- # 903-1 Any-Flex Bite 50ml x 4catridges
- # 903-2 Any-Flex Bite 50ml x 2catridges
- # 904-1 Any-Flex Clear 50ml x 4catridges

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Any-Flex Bite/Any-Flex Clear is a silicone-based occlusal material used to record the positional relationship between the maxilla and the mandible.

■ Indications

1. Impression of the occlusal surface
2. Check the occlusal surface
3. Intraoral incisal pin records after mounting of the occlude

■ Composition

Silicone-based material

■ Application method

1. Before the application
 - ① Check the expiration date and ensure the package is not contaminated.
 - ② Follow the instructions before using it.
 - ③ When ready to administer the procedure to the patient, attach the mixing gun to the product and dispense the

base and catalyst in a 1:1 ratio.

- ④ Be sure to wear respiratory protection before the procedure.
- ⑤ Attach the mixing tip to the cartridge.

2. Application

- ① Pull the handle of the mixing gun to dispense the material at a 1:1 ratio, and apply it to the occlusal or incisal surfaces of the lower teeth using the mixing tip.
- ② Apply an appropriate amount depending on the number of teeth.
- ③ Have the patient bite down on the applied material and hold for approximately 50 seconds.
- ④ Once the Any-Flex Bite/Any-Flex Clear is fully cured, carefully remove it from the mouth.
- ⑤ Check the occlusal registration.

■ Working Time / Setting Time

Product	Working time (23°C)	Curing time (32°C)	Hardness(Shore A)
Any-Flex Bite	0:20	0:30	91
Any-Flex Clear	0:30	2:00	60

■ Contraindications

Do not use Any-Flex Bite/Any-Flex Clear in patients with an allergy to Silicon-based material

■ Cautions

- 1. For dental use only.
- 2. The mixing time is included within the working time.
- 3. The working time may vary by approximately ±15 seconds for every 1°C change in temperature.
- 4. In case of contact with skin or eyes, rinse thoroughly with clean water and consult a specialist if necessary.
- 5. This product is prohibited for any use other than its intended purpose, and use by non-professionals is not allowed.
- 6. Follow the procedure and observe all precautions.
- 7. Keep out of reach of non-dental users, including children and the elderly.
- 8. Do not use after the expiration date.
- 9. Handle with care to prevent product damage, avoiding excessive impact.
- 10. Use of latex gloves may inhibit the setting of the impression material. It is recommended to use clean, washed hands or polyethylene gloves.
- 11. Residual astringents or hemostatic agents may interfere with

the setting. If hydrogen peroxide is used, rinse thoroughly with warm water to prevent foaming.

- 12. This product may cause permanent stains on clothing. Avoid contact with garments.

■ Storage

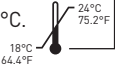
- 1. Avoid exposure to direct sunlight.
- 2. Store at a temperature between 18°C and 24°C.

■ Contents

- # 903-1 Any-Flex Bite 50ml x 4cartridges
- # 903-2 Any-Flex Bite 50ml x 2cartridges
- # 904-1 Any-Flex Clear 50ml x 4cartridges

■ Shelf-life

- 2 years from the date of manufacture

KTR EUROPE GMBH

Mergenthalerallee 77, 65760 Eschborn,
Germany
Phone : +49-6196-887170

SYMBOLS

: Use-by date	: Batch code
: Caution	: Keep Dry
: Manufacturer	: Date of Manufacture
: Authorized Representative in the European Community	
: Temperature limit	: Consult instruction for use
: Keep away from sunlight	: Catalogue number
: Model number	: Medical device
: Unique device identifier	: Non-sterile device
: Do not re-use	

■ MEDICLUS CO., LTD.

No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Republic of Korea
Tel. +82-43-211-2877 / Fax. +82-43-211-2866
E-mail : sales@mdclus.com