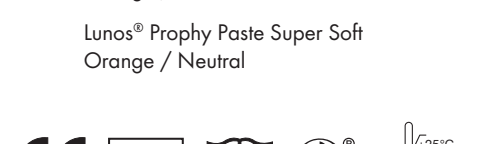
Lunos® Polierpaste Super Soft Orange / Neutral

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EN Instruction for use

Lunos® Prophy Paste Super Soft

Orange / Neutral
“Super Soft polishing paste” is available in the following variants:

Neutral (REF: CPZ740A0901), contains no flavouring agents

Orange (REF: CPZ750A0901), contains limonene

Product description
“Super Soft polishing paste” is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists. The formulation is free of animal ingredients, gluten, lactose, fructose and micro-plastics.

- its use is permissible only together with polishing instruments suitable for the prophylactic purpose
- for gentle removal of soft plaque coatings on supragingival tooth surfaces
- for polishing and high-gloss polishing of supragingival tooth surfaces
- for preparation and cleaning of tooth surfaces prior to adhesive bonding of orthodontic brackets, application of fluoride products, application of bleaching products, use of composite restoration materials and insertion of permanent prosthetics such as inlays, onlays, crowns, veneers
- for cleaning and polishing of the milk teeth set and on clinically accessible supragingival implant surfaces

Super Soft polishing paste has an RDA of < 5 and is suitable for children and adults provided a trained swallowing reflex is ensured.

Contraindications
“Super Soft polishing paste” are contraindicated in patients with known allergies to ingredients of the paste (e.g. limonene). If the patient has known allergies to limonene, the “Neutral” variant of “Super Soft polishing paste” is to be recommended.

Warning
Do not swallow. Avoid contact with the eyes. In case of contact with the eyes, rinse them plenty of water. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Precautions
Use suitable protective equipment when working with “Super Soft polishing paste”. Patients and operators need to wear suitable eye protection, e.g. protective goggles. The operator needs to wear suitable protective equipment in the form of mouth and nose protection and protective gloves.

“Super Soft polishing paste” may only be used by appropriately trained dental specialists.

Clean the surfaces of the dental unit with a moist cloth following contact with the polishing paste.

Preparation
For first application, the temper-end seal of the tube is to be removed. Squeeze out a sufficient amount of polishing paste into an application ring, e.g. Lunos® Prophylax Ring, dappen glass or similar suitable vessel. Take up polishing paste with a suitable polishing instrument and begin the polishing process. Please note that sufficient polishing past must be present between polishing instrument and tooth, in order to avoid generating heat on the tooth. For this reason, please also take care not to exceed the maximum speed of rotation recommended by the polishing instrument manufacturer. Please observe the directions for use of the device manufacturer.

Storage and shelf life
Please make sure that the tube is carefully closed again immediately after removal of paste, to prevent the paste from drying out.

Store in enclosed, well-ventilated rooms away from moisture and bright light between 0 °C and 25 °C. Keep out of direct sunlight. When the following storage conditions are observed, the shelf life is 36 months, or 12 months after opening.

Disposal
Dispose of in accordance with the local official regulations.

Composition
Super Soft Neutral Polishing Paste
Hydroxyapatite, glycerine, distilled water, thickener, preservative (phenoxylethanol, PHB ester), emulgator
Super Soft Orange Polishing Paste
Hydroxyapatite, glycerine, distilled water, thickener, preservative (phenoxylethanol, PHB ester), emulgator, orange flavouring agent (contains limonene), colouring agents (C.I. 15985, C.I. 16255)

* Method of Stookey/Schemohm using pumice as a reference substance for prophylaxis pastes (J Dent Res, 1979 58(2); p.588-592), relative RDA value as compared to product “P” = RDA 7, Measuring results: product “P” = 126 / Super Soft = 15

DÜRRENTAL, Products UK Ltd.
14 Lime Way – Telford Way, Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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