REF 316.62  
REF 316.62.09

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## DESCRIPTION

Rapid couplings for use with all quick-release TKD turbines or with any other dental instrument with KaVo Multiflex® connection (Fig. 1). Intended use: the device can be used for general dentistry works. Intended patient population: any person without restriction concerning age, gender, race or culture.

Patient contra-indications: there are no specific patient contra-indications or side effects, if the device is used as intended. Couplings are equipped with a non-return valve for water. Maximum operating pressure of air and water: 6 bar (87 psi). The rapid coupling is provided with an integrated LED (3.5 ± 0.3 Vdc; 60 mA maximum) for illumination.

This medical device meets the requirements of the Regulation (UE) MDR 2017/745 (Class I).

## WARNINGS

- Before using the device, read the following warnings carefully:
- The device must be used according to these operating instructions;
  - The device must be used by competent and qualified personnel only;
  - The device must be used only for the purpose for which it is intended;
  - The device does not produce electromagnetic interference.

## CONTENTS

GYROFLEX®LED rapid coupling, tool for coupling installation, set of spare O-rings.

## CONNECTION TO THE SUPPLY HOSE

Screw the coupling onto the flexible supply hose. The connector meets ISO 9168 Standard.

For a perfect locking, use and insert the supplied tool on the corresponding grooves (Fig. 2).

After that, the coupling does not need to be removed from the hose (Fig. 3). Couplings must be connected to the power supply unit meeting the requirements of CEI EN 60601-1 Standard.

Additionally, the power supply unit must have a minimum insulation strength of 500 V between the electric circuits and earth and a minimum insulation strength of 4000 V between the electric circuits and mains.

## SPRAY REGULATION (REF 316.62.09)

Rotate the spray ring on the coupling in order to regulate the fraction of water (Fig. 5). Different stop positions allow a precise regulation of the water volume.

Turn the ring clockwise to reduce the water volume. Turn the ring counterclockwise to increase the water volume.

## LED POSITIONING

The LED must be supplied with the right polarity. If the LED does not turn on: unscrew the protection cap (a), pull out the LED (b), rotate it by 180° so as to reverse polarity on its contacts and insert it again on its housing (c) (Fig. 4). Screw cap again.

## O.R. MAINTENANCE

In order to increase coupling seal and smoothness, we recommend that you regularly rub the O-rings with a small quantity of non-toxic grease. The grease can also be taken from the turbine grease lubricator (LubriONCE®)

## O.R. REPLACEMENT

Should water or air leakage occur, replacement of O-rings on the coupling is recommended.

With the aid of a pin, pull out the old O-rings and insert the new ones on the corresponding grooves (Fig. 6).

## CLEANING AND DISINFECTION

Couplings can be externally disinfected with a *non-chlorinated* germicidal cleanser. Additionally, the cleanser must not contain any *glutaraldehyde, acid or solvent*. Rapid couplings cannot be sterilized.

## GUARANTEE

The device carry a 24-month guarantee against all defects of construction. The guarantee does not cover the LED.

## INCIDENTS

If any serious incident occurs in relation to the device, report it to a competent national authority, as well as to the manufacturer through your usual distributor.

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Multiflex® is a registered trademark of Kaltenbach & Voigt GmbH, Germany.