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One-Stop is a paste-type dental pre-treatment material applied into the gingival sulcus to remove blood and crevicular fluid, and to temporarily expand the sulcus. It is intended to assist in obtaining accurate and clear dental impressions.

■ Indications

1. Local hemostasis and moisture control in the gingival sulcus
2. Pre-treatment prior to prosthetic impressions or restorations
3. Prior to core buildup for gingival retraction
4. To secure clear visibility before adhesive procedures

■ Composition

An aluminum chloride-based astringent agent

■ Application method

1. Before the application
 - ① Check the expiration date and ensure the package is not contaminated.
 - ② Follow the instructions before using it.
 - ③ When ready to administer the procedure to the patient, remove the cap of the product.

④ Be sure to wear respiratory protection before the procedure.

2. Applications

1) Check application site

- ① Assess the condition of the gingival sulcus and surrounding tissues.
- ② If necessary, gently remove debris, blood, or fluid to clean the site.
- ③ Ensure that the application site is free of active infection, severe inflammation, or ulcers.

2) Application

- ① Slowly apply One-Stop using the tip to the desired are.
- ② Apply enough material to fill the gingival sulcus.
Avoid excessive injection.
- ③ Add a small amount if necessary to ensure even coverage.

3) Application time

Leave the product in place for approximately 2–3 minutes.
During this time, the material absorbs fluid and minor bleeding, and induces temporary gingival sulcus expansion via astringent action.

4) Removal

- ① After 2–3 minutes, rinse thoroughly with clean water or saline.
- ② Use gentle water pressure to ensure complete removal of any remaining material.
- ③ If residue remains, remove with a suction tip or air-water spray.

5) Post-procedure

- ① After rinsing is complete, proceed with impression taking.
- ② Reassess the oral cavity to ensure there is no mucosal damage or hypersensitivity.
- ③ Do not reuse the syringe or tip. Dispose of used items as medical waste.

■ Contraindications

1. Do not use in patients with hypersensitivity or a history of allergic reactions to any components of the product (e.g., aluminum chloride).
2. Do not apply to areas with active infections, severe periodontitis, gingival ulcers, or open wounds.
3. Do not use in children under 3 years of age or in patients with significantly compromised immune function.
4. Reuse of the enclosed tip may cause cross-infection, so the reuse of the tip is prohibited.

■ Cautions

1. For dental use only.
2. Do not use in patients with a known history of hypersensitivity such as rash or dermatitis caused by this product.
3. This product is intended for use only on oral tissues.
4. If hypersensitivity symptoms such as rash occur during use, discontinue use and consult a dermatologist.
5. This product is prohibited for any use other than its intended purpose, and use by non-professionals is not allowed.
6. Follow the procedure and observe all precautions.

7. Keep out of reach of non-dental users, including children an elderly.
8. Always check the lot number and expiration date before use.
9. Handle with care to prevent product damage, avoiding excessive impact.
10. Use of this product other than components is prohibited.

※ After each use, the dispensing tip must be removed and replaced with a new tip before further application.


■ Adverse reactions

Adverse reactions may include, but are not limited to :

1. Mild stinging, burning sensation, or temporary irritation.
2. Allergic reactions such as rash, swelling, or difficulty breathing discontinue use immediately and consult a medical professional.
3. Excessive application or residue may cause mucosal damage or secondary infection.

■ Storage

1. Avoid exposure to direct sunlight.
2. Store at temperature between 1°C and 30°C.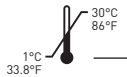

■ Contents

- # 1101-1 One-Stop Blue 0.85g x 7syringes, Disposable tips
- # 1101-2 One-Stop Blue 0.85g x 2syringes, Disposable tips

■ Shelf-life

2 years from the date of manufacture

SYMBOLS

: Use-by date	: Batch code
: Caution	: Keep Dry
: Manufacturer	: Date of Manufacture
: Authorized Representative in the European Community	
: Temperature limit	: Consult instruction for use
: Keep away from sunlight	: Catalogue number
: Country of manufacture	: Model number
: Unique device identifier	: Medical device
: Do not re-use	: Non-sterile device

■ MEDICLUS CO., LTD.

No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Republic of Korea
Tel. +82-43-211-2877 / Fax. +82-43-211-2866
E-mail : sales@mdclus.com